

HIT-W183/W153 Quick Start Manual

Packing List

Before installing the system, the user must confirm that the package contains equipment and the following items::

- 1 x HIT-W183/153
- 1 x Accessory box
 - 1 x This quickstart manual
 - 1 x Adaptor
 - Bracket: Fix Adaptor
- China RoHs

If any of these items are missing or damaged, contact your dealer or sales representative immediately.

HIT-W183/153 Front/Rear View






Front



Rear

* Use only VESA-compliant mounting brackets.

HIT-W183/153 Overview

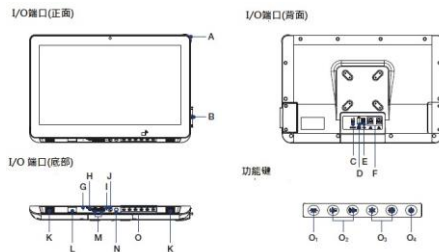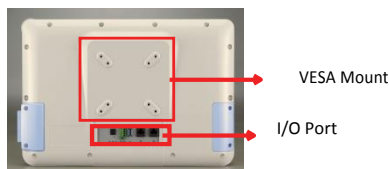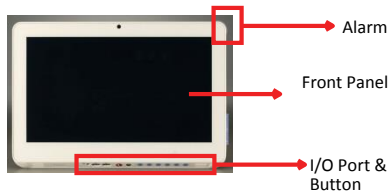

For more information about this product and Advantech's other products, visit our website:

<http://www.advantech.com.tw/support>

<http://www.advantech.com>

For technical service and support, please visit our technical support website:

<http://www.advantech.com/support>

- | | |
|------------------------------------|--------------------------------|
| A.Alarm Light | K.Speaker |
| B.Smart Card Reader | L.Barcode Reader (Optional) |
| C.Power-in DC Jack | M.USB 2.0 Port x2 |
| D.Nursing Call Jack | N.Reading Light |
| E.USB 3.0 Port x2 | O.Function Button |
| F.RJ-45 Portx2 | O1.Reading Light On/Off |
| G.Mic | O2.Volume Control Up/Down |
| H.Earphone | O3. Brightness Control Up/Down |
| I.Reset Button | O4.LCD Panel Control On/Off |
| J. Nursing Call Button (Emergency) | |

Easy Setup

Turn on computer power

1. Make sure the "equipotential terminal" is connected to the equipotential ground of the hospital
2. Connect the adapter to the power input jack. Please note that using non-genuine adapters may pose potential risks to the system
3. When you plug in the DC power, the HIT will power on directly.
4. DC Input Voltage: The adapter only supply 18-19Vdc

Install the Driver

This product has an operating system and drivers installed

If you have trouble installing the operating system, follow the instructions in the HIT-W183 Instruction Manual.

Note: Please use the "original genuine operating system" state to install this automatic installer; otherwise, an unexpected error may occur. If you only want to install a specific driver, follow the instructions in the HIT-W183 Instruction Manual.

Software Specifications

Please always check Advantech website for free downloading:

http://support.advantech.com/support/new_default.aspx

Cleaning and Disinfecting

During normal use, the HIT (Healthcare Information Terminal) may become dirty and should be cleaned regularly

1. Prepare a cleaning agent per manufacturer's instruction or hospital protocol.
2. Prepare a clean cloth that has been moistened in a cleaning solution.
3. Wipe the HIT thoroughly with a clean wipe.

Applicable Cleaning Agents List

Number	Detergent	Number	Detergent
1	Glutaraldehyde	11	Virkon and Water (1:100)
2	Isopropyl alcohol	12	Dispatch Hospital Detergent
3	Soft SOAP Tincture	13	Trigene Disinfectant
4	Windex	14	SDW 70C
5	Alcohol	15	Antigone Wipes
6	70% alcohol	16	Sporeclear
7	Incidin Plus Disinfectant	17	Antigone
8	Incidin Disinfectant	18	Mikrobac
9	Mikrocid Disinfectant		
10	1000PPM Bleach		

- * Do not immerse or rinse the HIT or its peripherals.
- * Do not spray cleaning agents on the chassis
- * Do not use disinfectants containing phenol